




## Medication Carry Case Model MCC-SM

### Setting the Combination Lock

Your Medication Carry Case has been set with a default security code of (0-0-0) from the factory. You may keep this code, however, for added security, we recommend that you re-set the combination using 3-digits of your choosing by using the following instructions:

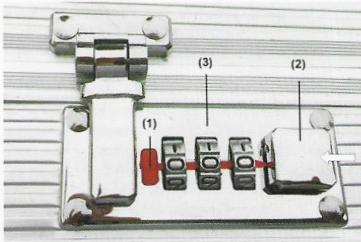

- Step #1 Remove the red plastic protective insert (1) from the combination lock.
- Step #2 Press the open button (2) to release the lock.
- Step #3 With the lock open, Press the open button (2) in the direction of the arrow and hold down until the second operation has been performed.
- Step #4 Set number wheels (3) to the desired combination.
- Step #5 Now release the button (2). Press the button again to check.
- Note: Make a note of your personal combination and keep it at a safe place!  
You will only be able to reset the combination when the lock is open.