

Reverso en español



INSTRUCTIONS FOR USE:

Easy to Install:

- With latches up, position Safe-er-Grip™ where desired on tub or shower wall.
- Apply pressure to the handle while flipping the latches down.
- Be sure to attach to smooth dry surfaces only.

IMPORTANT:

Check Safe-er-Grip regularly to ensure it is still firmly attached. If it can be moved or feels at all loose, flip up the latches and reinstall as per instructions above.

Easy to Remove:

- Release both latches.
- Slide a fingertip under rubber seal at each end to release suction. If necessary, slide something flat, such as a credit card, under the suction cups to release grip.



CAUTION: Safe-er-Grip is to be used as a balance assist only. Not for body weight leverage – body weight is not to be applied to product. (If device is required to bear any body weight, or is used for leverage, it is recommended that a traditional permanently mounted grab bar be installed.) Check attachment before each use and reattach if loose.

Style #40524-#01067
Made in China





INSTRUCCIONES DE USO:

Fácil de instalar:

- Coloque el Safe-er-Grip™ donde desee, en la pared de la bañera o la ducha, con los pestillos orientados hacia arriba.
- Presione la manija mientras empuja los pestillos hacia abajo.
- Asegúrese de fijarlo en superficies lisas y secas únicamente.

IMPORTANTE:

Revise el Safe-er-Grip regularmente para asegurarse de que aún se encuentre bien sujeto. Si se puede mover o está flojo, mueva los pestillos hacia arriba y vuelva a colocarlo según las instrucciones anteriores.

Fácil de retirar:

- Destrabe ambos pestillos.
- Para liberar las ventosas, deslice la punta del dedo por debajo del sellado de goma ubicado en cada extremo. Si es necesario, deslice algo plano, como una tarjeta de crédito, por debajo de las ventosas para liberar el agarre.



PRECAUCIÓN: Safe-er-Grip se debe usar sólo como ayuda para mantener el equilibrio. No se debe usar para sostener el peso del cuerpo; no se debe aplicar el peso del cuerpo sobre el producto. (Si el dispositivo necesita soportar el peso del cuerpo o ser utilizado como palanca, se recomienda instalar una barra de agarre tradicional de forma permanente). Revise la sujeción antes de cada uso y vuelva a instalarlo si se encuentra flojo.

Estilo #40524-#01067
Hecho en China

